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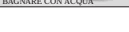
POSIZIONARE

ASCIUGARE

2 GB Information relating to supplementary long-sight OPTX 20/20

INSTRUCTIONS RELATING TO PRECAUTIONS NOTES FOR USE

OPTX 20/20 are supplementary lenses, with the same dioptric rating, recommended EXCLUSIVELY to correct simple long-sightedness and intended only for seeing things in close proximity or for reading. For such reasons they are not designed for the vision of distant objects. The OPTX 20/20 lenses should not be used in the case of 'complex' long-sightedness. Such lenses are not designed to substitute made-to-measure solutions but to correct selective vision problems. The OPTX 20/20 lenses can be mounted only on single protection accessories (such as sunglasses) and medical accessories (such as first aid category frames). They should not be used for protection accessories either in second or third categories (such as filters for welding goggles, motor cycling goggles or other protective masks).

The OPTX 20/20 are not intended to be used when driving a car or motor vehicle. Keep OPTX 20/20 lenses out of the reach of young children.

CLEANING AND CARE

Do not clean lenses with Spirit or Solvents. After use, the OPTX 20/20 lenses can be removed (as described in the instructions for use) and stored in their original packaging. Before reuse, first clean the lenses with water. Make sure the contact surface of the lenses appears to be scratched or opaque, substitute them with a new pair of lenses. If after a certain length of time the OPTX lenses lose their original characteristics of flexibility and softness, at this point they should be replaced with a new pair of lenses.

NOTE

To enable an accurate evaluation of the nature of the sight defect, it is necessary to be examined by a qualified professional, self-diagnosis is not sufficient. The use of incorrect or the wrong strength of OPTX lenses may cause various visual disturbances, with possible blindness. The supplementary long-sight OPTX lenses do not improve your vision if it is due to an eye disease, such as cataract or macular degeneration.

ATTENTION

- Only for viewing things in close proximity and reading.

- Only by visiting your qualified professional can you determine your visual requirements and assess the health of your eyes.

- Not for use while driving a car or motor vehicle.

- Not for use when driving a motor vehicle.

INSTRUCTIONS FOR USE

1. Wash the lenses of your glasses in soapy water and then dry.
2. Place a drop of water inside the lens of your glasses (where you are going to apply the OPTX lens).
3. Carefully remove the OPTX lenses from their original case and handle with care.

4. Take an OPTX lens between your thumb and index finger and place it on the inside of the glass lens, positioning it at the lowest part of the lens, near your nose (OPTX lenses can easily move whilst moist). Repeat this operation with the second lens, positioning it in the same way as before, ensuring both lenses are symmetrical in relation to your nose.

CAUTION: Make sure that the curvature of the OPTX lenses is oriented as the curvature of the lenses of the glasses.

5. With a dry cloth gently press the OPTX lenses in a way that forces out excess water.
6. After approximately half an hour you can use the glasses (the dry OPTX lenses remain attached due to molecular adhesion). To remove the lenses simply lift the edges.
7. Put the glasses on and check that your direct line of vision is straight and not obstructed by the OPTX lenses. If your line of vision is hindered, remove the lenses as described in point 6 and cut a parallel section from the largest horizontal side, to a maximum of 5mm. Reposition the modified lenses as described in point 1 to 6 and check again, wearing the glasses.
8. Wearing the glasses, look downwards (at the level of the OPTX lenses) and check that close vision is of optimum clarity and they do not cause a nuisance or result in tiredness of vision. If they do, reposition the lenses very lightly in the same approximate position as described in points 1 to 6. If you are still unable to achieve optimum results after repositioning the lenses, utilise the OPTX lenses with a different dioptric rating or consult a specialist.

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E.A.R. Obelis, S.r.l.
MADE IN THE U.S.A.
U.S.A. & EUROPEAN PATENT